- EN** Unheated disposable breathing tube system
- DE** Unbeheiztes Einweg-Atemschlauchsystem
- FR** Circuit de tuyaux respiratoires, sans chauffage, non réutilisable
- IT** Circuito respiratorio monouso, non riscaldato
- ES** Sistema de tubo de respiración monouso, sin calor
- NL** Niet verwarmd ademhalingsstelsel voor eenmalig gebruik
- GR** Μη θερμαινόμενο κύκλωμα αναπνευστήρα μιας χρήσης
- TR** Isıtılmayan tek yönlü solunum tüpü sistemi
- PL** Nieogrzewany system węży oddechowych jednorazowego użytku
- SV** Ouppvärmt engångssystem för andnings slang

Key to Symbols / Zeichenerklärung / Signification des Symboles / Spiegazione dei Simboli / Significado de los Simbolos / Verklaring der Tekens / Υπόμνημα / İşaret açıklaması / Objasnienie znaków / Teckenförklaring

REF	LOT	
Re-order number Bestellnummer No. de Référence Codice di articolo Número de referencia Bestelnummer Αριθμός παραγγελίας Sipariş numarası Numer zamówieniowy Beställningsnummer	Lot number Chargennummer Numéro de lot Lotto Número de lote Chargennummer Αριθμός παρτίδας Parti no Numer partii Partiets nummer	Use by date Verfalldatum À utiliser avant Data di scadenza Fecha de caducidad Vervaldatum Ημερομηνία λήξης Dava tarihi Data ważności Utgångsdatum
		
Date of manufacture Herstellungsdatum Date de fabrication Data di produzione Fecha de fabricación Productiedatum Ημερομηνία κατασκευής Üretim tarihi Data produkcji Tillverkningsdatum	Manufacturer Hersteller Producteur Produttore Fabricante Producent Κατασκευαστής Yapımcı Producent Tillverkare	Latex free Latex frei Sans latex Senza lattice Sin latex Latexvrij Χωρίς λάτεξ Lateks içermez Nie zawiera lateksu Latex-fri

Disposable: for 1 patient, usable 7 days max./Einweg: für 1 Patient, max. Anwendung 7 Tage/A usage unique: pour 1 patient, utilisation pendant 7 jours max./Monouso: per 1 paziente, durata di utilizzo max. 7 giorni/Desechable: para 1 paciente, duración máx. de la aplicación 7 días/Eenmalig: voor 1 patiënt, max. gebruiksduur 7 dagen/Μίας χρήσης: για 1 ασθενή, μέγ. χρόνος εφαρμογής 7 ημέρες/Tek kullanımlık: 1 hasta için, en fazla 7 gün kullanılabilir/Jednorazowego użycia: dla 1 pacjenta, max. czas stosowania 7 dni/Engångs: till 1 patient, längsta användning 7 dagar

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Application

This product is intended for the conveyance of humidified or not humidified respiratory gas from the therapy device to the patient.

Instructions for use

• Connection and disconnection of tube is effected by turning, pressing or pulling motions at the sleeve – not at the tube itself. • Connect therapy ce and input of humidifier chamber with short tube (if existent). • Connect out of humidifier chamber with inspiratory tube. • Connect possible corollary (e.g. pressure line) to the therapy device. • In case of a double limb em, connect expiratory side of the breathing tube system to the therapy ce. • Align water trap (if existent) so that reservoir points downward. • Before starting operation please check system for blockages, leakages, and connections. • At first put therapy device into operation, then the humidifier. • Slight condensation inside breathing tube and humidifier chamber during operation is normal. • Check water traps (if existent) regularly, empty out water cessary. Then check if water trap is correctly closed, again.

Warnings

• The Y-piece or exhalation valve (if existent) must neither be closed nor used. • This product is destined for one single patient and is not intended for re-use (cleaning with a term of usage of max. 7 days (do not utilize again after use, do not clean or sterilize it). • Make sure that this breathing tube system is appropriate for the intended use and for the provided device. • Warranty expires when the package is opened. • Store in a cool dry place. Do not expose to sunlight. • Do not keep or operate it close to heat sources. • After use, dispose in accordance to regulations for infectious waste.

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WILAMED GmbH